Lactate SCOUT

(IT)

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Lactate SCOUT

(UK)

Product information about the Lactate Scout-test strips for one-time disposable use with the Lactate Scout analyzer to determine lactate in whole blood.
Please read carefully before using!

INTENDED USE

Lactate Scout test strips are only to be used with the Lactate Scout analyzer to determine the lactate concentration in capillary blood. For in-vitro-application only! The Lactate Scout test strips and the Lactate Scout analyzer can be used both by individuals and by health personnel.

WARNING NOTICES

Read the instruction manual and package insert before using the Lactate Scout! Avoid any risk of test strips being swallowed! Keep away from children! Always wear gloves when you are carrying out a measurement on another person! The handling with blood carries generally the risk of infections: Please make sure items containing or contacted by blood (such as test strips, lancets and alcohol swabs) are disposed of properly according to the local regulations, to avoid potential abuse, contamination and other risks to third parties.

STORAGE

Keep the test strips protected from moisture, light, and temperatures above +50°C / 122°F! Keep refrigerated! Transport and store the test strips only in their original vial! Take out the required test strips for immediate use only and close the vial again, to keep the other test strips protected continually. Before using refrigerated test strips, take the vial out and keep it at regular temperature for 20 min at least: When used, analyzer and test strips should be at same temperature. After first opening the vial of test strips, the storage stability is three months or until expiry date if it is sooner. Afterwards, inaccurate measurement results will occur increasingly.

EVEN READINGS / INTERFERENCES

Wrong little rests of sweat, saliva, desinfection or alcohol on the skin surface may cause problems of sample uptake or leads to wrong readings (as too high results): Ensure that the area (fingertip or earlobe) is cleaned carefully by water and dried before each sampling again. Hurry up with the sample uptake after puncturing: Only fresh capillary blood can be used for meaningful results!

Also taken drugs (as paracetamol) as well as illness, infection, diet, fat meals, physical or psychical stress may influence the measurement results differently.

HANDLING (ONE-TIME DISPOSABLE USE)

After cleaning and puncturing of the sampling area, a new, non-used Lactate Scout test strip has to be inserted into the Lactate Scout analyzer. Therefore, hold the test strip in the middle and do not touch the tip! The hands should be clean and dry. By inserting the test strip, the calibration code will be displayed for 1 s: Ensure that it is identical to the code as printed on the currently used vial of test strips. Also control the expiry date!

Lower the Lactate Scout analyzer with the tip of the inserted test strip from top down to the fresh drop of blood (if possible, wipe away the first drop before and take the second one). Avoid smearing the blood sample, and do not touch the skin! The required sample volume (0.5 µl) will be taken automatically: When the display shows LAC, the measurement runs and the result will be shown after 15 s. By removing the test strip, the result will be stored automatically (incl. date, time and other data), and the Lactate Scout analyzer switches off.

Error messages: By readings above 25.0 mmol, „Hi“ appears, and „Lo“ by readings lower than 0.5 mmol/l. By usability problems or malfunctions, an error message will appear („Er 1-6“, as explained in the manual). If the temperature is lower than +5°C / 41°F or higher than +45°C / 113°F, a warning symbol appears and no measurement occurs.

Health note: There are no known health risk from using Lactate Scout test strips according to the instructions: SensLab products are not authorized for use as critical components in life support devices or systems! Persons prone to excessive bleeding or infection, or persons feeling ill have to consult a doctor generally before puncturing and sampling.




TECHNICAL DATA

Measurement method: Enzymatic-amperometric detection of lactate
Reagent per test strip: Lactate oxidase, electron mediator, additives
Coefficient of variation: 3.0 - 8.0% (depends on concentr.; validated average)
Sample volume: 0.5 µl **Measuring range:** 0.5 - 25.0 mmol/l

TEST SOLUTION (option):

Contains buffer solution, additives and liquid with a lactate concentration of 9,5 - 12,5 mmol/l. The test solution is suitable only for a general function control without any calibration! Test solutions are delivered optionally e.g. as part of the „Lactate Scout BigPack 48+“. Other calibration solutions, liquids or control materials cannot be used properly with Lactate Scout!

For additional information visit: www.lactatescout.com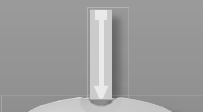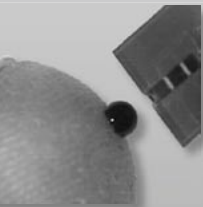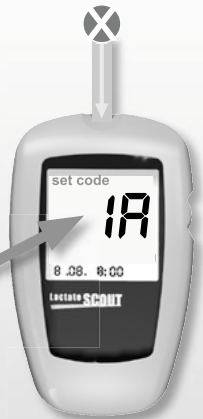

Distribution:
EKF - diagnostic sales GmbH
Ebenendorfer Chaussee 3
D - 39179 Barleben/Magdeburg
Tel. +49 (0) 39 203 - 785 - 0
Fax: +49 (0) 39 203 - 785 - 24
sales@ekf-diagnostic.de

Hersteller/Manufacturer:
SensLab GmbH
Bautzener Str. 67, D - 04347 Leipzig
Tel. +49 (0) 341 - 234 18 30
Fax: +49 (0) 341 - 234 18 40
info@senslab.de

www.lactatescout.com